

Zertifizierter taktischer Hersteller von Turnierkerngültigkeiten für Industriestärke

Produktdetail

Produktname	Tourniquet
Farbe	Schwarz
Größe	3,8*65 cm, 3,8*75 cm, 3,8*95 cm
Material	Hakenschleifenband und Plastik/mentale Stange
MOQ	Kleine Bestellung akzeptieren
Paket	1pc/Opp -Tasche oder individuelles Paket
Anpassung	Das benutzerdefinierte Logo, die Verpackung und die Farbe können gemäß den Benutzeranforderungen erfolgen
Funktion	Mit einer Hand angewendet, schnell und praktisch, zufällige Kratzer, Schnitte und andere gute Auswirkungen der Hämostase.

Tourniquet ist eine häufige lebensrettende Ausrüstung, die aus hochwertigen Materialien besteht und mit einer Hand betrieben werden kann, wodurch der Blutfluss vollständig an den Gliedmaßen durch schwere Blutungsverletzungen blockiert wird. Es ist ein großartiges Erste -Hilfe -Tool mit einer Vielzahl von Anwendungsszenarien, nicht nur in Kampf- und medizinischer Rettung, sondern auch in Außen-, Abenteuer- und arteriellen Blutungen. Ein Muss für alle!



CAT ROTARY TOURNIQUET



Easy to use



Breathable
&
Comfortable



Effective
Hemostasis



Compact
&
Convenient





Hook loop tape

TIME:

Time recording tape

Pressure fixed rod

Single Route Buckle

FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response



RECORD USAGE TIME

Write down the time of use to easily determine the length of time



TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control



PRODUCT USAGE



Wilderness
Combat



Outdoor
Adventure

PREPAREDNESS AVERTS PERIL



Bleeding
from injury



Hiking and
camping

— HOW TO USE —



01
Tie the tourniquet three fingers' width above the wound to stop bleeding



02
Apply tourniquet and tighten to stop bleeding



03
Pull the hook loop tape tight and paste it back on



04
Rotate the pressure fixed rod to tighten the strap



05
Fasten enough to use the clasp to fix the rod



06
Check for bleeding and distal pulse loss



07
Attach the white hook loop tape



08
Write down the time to prevent muscle necrosis caused by too much time

Unternehmensinformationen

CONTACT INFORMATION

TEL : +80 20 34992808
+86 18026363923

Web : www.homecommoditysupplier.com

E-mail : sales02@balance-china.com

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470