

# Torniquete certificado Distribuidor mayorista Fabricante táctico de fuerza industrial

## Detalle del producto

<b>Nombre del producto</b>	Torniquete
<b>Color</b>	Negro
<b>Tamaño</b>	3.8*65cm, 3.8*75cm, 3.8*95cm
<b>Material</b>	Cinta de bucle de gancho y barra de plástico/mental
<b>Moq</b>	Acepta un pedido pequeño
<b>Paquete</b>	1pc/opp bag, o paquete personalizado
<b>Personalización</b>	El logotipo personalizado, el embalaje y el color se pueden hacer de acuerdo con los requisitos del usuario
<b>Función</b>	Aplicado con una mano, raspados accidentales rápidos y prácticos, cortes y otros buenos efectos de la hemostasia.

**El torniquete es un equipo común que salva la vida, que está hecho de materiales de alta calidad y se puede operar con una mano, bloqueando completamente el flujo sanguíneo a las extremidades de las heridas graves de trauma hemorrágicas. Es una gran herramienta de primeros auxilios con una amplia gama de escenarios de aplicación, no solo en combate y rescate médico, sino también en sangrado al aire libre, aventura y arterial. ¡Un imprescindible para todos!**

---



# CAT ROTARY TOURNIQUET



Easy to use



Breathable  
&  
Comfortable



Effective  
Hemostasis



Compact  
&  
Convenient





**Hook loop tape**

**TIME:**

**Time recording tape**

**Pressure fixed rod**

**Single Route Buckle**

# FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



# HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response



# RECORD USAGE TIME

Write down the time of use to easily determine the length of time



# TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control



# PRODUCT USAGE



Wilderness  
Combat



Outdoor  
Adventure

## PREPAREDNESS AVERTS PERIL



Bleeding  
from injury



Hiking and  
camping

# — HOW TO USE —



**01**  
Tie the tourniquet three fingers' width above the wound to stop bleeding



**02**  
Apply tourniquet and tighten to stop bleeding



**03**  
Pull the hook loop tape tight and paste it back on



**04**  
Rotate the pressure fixed rod to tighten the strap



**05**  
Fasten enough to use the clasp to fix the rod



**06**  
Check for bleeding and distal pulse loss



**07**  
Attach the white hook loop tape



**08**  
Write down the time to prevent muscle necrosis caused by too much time

## Información de la empresa

# CONTACT INFORMATION

TEL : +80 20 34992808  
+86 18026363923

Web : [www.homecommoditysupplier.com](http://www.homecommoditysupplier.com)

E-mail : [sales02@balance-china.com](mailto:sales02@balance-china.com)

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,  
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470