

Gourse de gros professionnel pour les premiers soins et l'alimentation des fournitures médicales d'urgence

Détail du produit

Nom de produit	Tourniquet
Couleur	Noir
Taille	3,8 * 65cm, 3,8 * 75cm, 3,8 * 95cm
Matériel	Ruban de boucle à crochet et canne en plastique / mental
MOQ	Accepter la petite commande
Emballer	1 sac PC / OPP, ou emballage personnalisé
Personnalisation	Le logo personnalisé, l'emballage et la couleur peuvent être fabriqués en fonction des besoins des utilisateurs
Fonction	Appliqué d'une main, rapide et pratique, éraflures accidentelles, coupes et autres bons effets de l'hémostase.

Le garrot est un équipement de sauvetage commun, qui est composé de matériaux de haute qualité et peut être utilisé d'une main, bloquant complètement le flux sanguin vers les membres à partir de plaies de traumatisme saignant sévères. C'est un excellent outil de premiers soins avec un large éventail de scénarios d'application, non seulement dans le combat et le sauvetage médical, mais aussi en extérieur, aventure et saignement artériel. Un incontournable pour tout le monde!

CAT ROTARY TOURNIQUET



Easy to use



Breathable
&
Comfortable



Effective
Hemostasis



Compact
&
Convenient





FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response



RECORD USAGE TIME

Write down the time of use to easily determine the length of time



TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control



PRODUCT USAGE



**Wilderness
Combat**



**Outdoor
Adventure**

PREPAREDNESS AVERTS PERIL



**Bleeding
from injury**



**Hiking and
camping**

— HOW TO USE —



01
Tie the tourniquet three fingers' width above the wound to stop bleeding



02
Apply tourniquet and tighten to stop bleeding



03
Pull the hook loop tape tight and paste it back on



04
Rotate the pressure fixed rod to tighten the strap



05
Fasten enough to use the clasp to fix the rod



06
Check for bleeding and distal pulse loss



07
Attach the white hook loop tape



08
Write down the time to prevent muscle necrosis caused by too much time

Information de l'entreprise



CONTACT INFORMATION

TEL : +80 20 34992808
+86 18026363923

Web : www.homecommoditysupplier.com

E-mail : sales02@balance-china.com

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470