

CAT

--	--	--	--	--

[illegible]

CAT

CAT ROTARY TOURNIQUET



Easy to use



Breathable
&
Comfortable



Effective
Hemostasis



Compact
&
Convenient





FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response



RECORD USAGE TIME

Write down the time of use to easily determine the length of time



TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control



PRODUCT USAGE



**Wilderness
Combat**



**Outdoor
Adventure**

PREPAREDNESS AVERTS PERIL



**Bleeding
from injury**



**Hiking and
camping**

— HOW TO USE —



01
Tie the tourniquet three fingers' width above the wound to stop bleeding



02
Apply tourniquet and tighten to stop bleeding



03
Pull the hook loop tape tight and paste it back on



04
Rotate the pressure fixed rod to tighten the strap



05
Fasten enough to use the clasp to fix the rod



06
Check for bleeding and distal pulse loss



07
Attach the white hook loop tape



08
Write down the time to prevent muscle necrosis caused by too much time



We have 5000m² factory, our own factory
The REAL STRENGTH is to show the true face to you





Strict certification testing provides quality guarantee and **WORRY-FREE EXPORT**



We have achieved some inspection certification, such as ISO9001:2015, BSCI, SGS, TUV, etc. and have won the title of "Guangdong Province Credible Enterprise" for three consecutive years. In addition, we have obtained over 20 product patents and trademark, as well as 5 patent evaluation reports. A series of quality inspection reports provide quality guarantee and worry-free export.



CONTACT INFORMATION

TEL : +80 20 34992808
+86 18026363923

Web : www.homecommoditysupplier.com

E-mail : sales02@balance-china.com

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470