

1. 项目背景与意义

2. 项目目标

项目背景	项目背景
项目目标	项目目标
项目范围	3.8*65cm 3.8*75cm 3.8*95cm
项目周期	项目周期
项目预算	项目预算
项目风险	项目风险
项目成果	1. PC/OPP 项目成果
项目总结	项目总结
项目附件	项目附件

3. 项目组织与分工

4. 项目进度计划

CAT ROTARY TOURNIQUET



Easy to use



Breathable
&
Comfortable



Effective
Hemostasis



Compact
&
Convenient





FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response



RECORD USAGE TIME

Write down the time of use to easily determine the length of time



TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control



PRODUCT USAGE



**Wilderness
Combat**



**Outdoor
Adventure**

PREPAREDNESS AVERTS PERIL



**Bleeding
from injury**



**Hiking and
camping**

— HOW TO USE —



01
Tie the tourniquet three fingers' width above the wound to stop bleeding



02
Apply tourniquet and tighten to stop bleeding



03
Pull the hook loop tape tight and paste it back on



04
Rotate the pressure fixed rod to tighten the strap



05
Fasten enough to use the clasp to fix the rod



06
Check for bleeding and distal pulse loss



07
Attach the white hook loop tape



08
Write down the time to prevent muscle necrosis caused by too much time

00000

CONTACT INFORMATION

TEL : +80 20 34992808
+86 18026363923

Web : www.homecommoditysupplier.com

E-mail : sales02@balance-china.com

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470