


	CAT [][] [][]
	3.8*65cm, 3.8*75cm, 3.8*95cm
MOQ	
	100 PC/opp 00, 00 000 000 00
	0 000 0000 000 0000 0000 00, 00 0 00 00

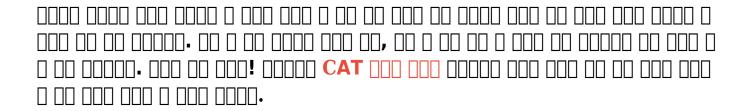

CAT ROTARY TOURNIQUET





FREE ADJUSTING BUCKLE



HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response



RECORD USAGE TIME

Write down the time of use to easily determine the length of time



TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control





HOW TO USE-



Tie the tourniquet three fingers' width above the wound to stop bleeding



Apply tourniquet and tighten to stop bleeding



Pull the hook loop tape tight and paste it back on



Rotate the pressure fixed rod to tighten the strap



Fasten enough to use the clasp to fix the rod



Check for bleeding and distal pulse loss



Attach the white hook loop tape



Write down the time to prevent muscle necrosis caused by too much time



























Strict certification testing provides quality guarantee and WORRY-FREE EXPORT



We have achieved some inspection certification, such as ISO9001:2015, BSCI, SGS, TUV, etc. and have won the title of "Guangdong Province Credible Enterprise" for three consecutive years. In addition, we have obtained over 20 product patents and trademark, as well as 5 patent evaluation reports. A series of quality inspection reports provide quality guarantee and worry-free export.

CONTACT INFORMATION

:+80 20 34992808 +86 18026363923 TEL

: www.homecommoditysupplier.com Web

E-mail :sales02@balance-china.com

Address: No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town, Nansha District, Guang zhou, Guang dong, China

Zip code: 511470