

□□ □□□

□□.□□.□□

□□□	CAT □□□ □□□
□□	□□□
□□	<b>3.8*65cm, 3.8*75cm, 3.8*95cm</b>
<b>MOQ</b>	□□ □□ □□
□□□	<b>1□□ pc/opp</b> □□, □□ □□□ □□□ □□□ □□
□□□□□□	□□□ □□ □□, □□ □ □□□ □□□ □□ □□□ □□ □□ □ □□□□.
□□	□ □□□ □□□ □□□□□ □□□ □ □□, □□ □ □□ □□ □□□ □□□□.

□□□□ □□□□ □□ □□□ □□ □□□ □□□ □ □□□ □□□ □ □□ □□□ □□ □□□ □□□ □□ □□□ □□□□. □□ □ □□ □ □□□ □□□ □□, □□ □ □□ □□□□□ □□□□ □□ □□□□□ □□ □□□ □□ □□ □□□□□. □□□ □□ □□ □□□!



# CAT ROTARY TOURNIQUET



Easy to use



Breathable  
&  
Comfortable



Effective  
Hemostasis



Compact  
&  
Convenient





**Hook loop tape**

**TIME:**

**Time recording tape**

**Pressure fixed rod**

**Single Route Buckle**

# RECORD USAGE TIME

Write down the time of use to easily determine the length of time



# TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control



# PRODUCT USAGE



**Wilderness  
Combat**



**Outdoor  
Adventure**

## PREPAREDNESS AVERTS PERIL



**Bleeding  
from injury**



**Hiking and  
camping**

# FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



# HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response





# — HOW TO USE —



**01**  
Tie the tourniquet three fingers' width above the wound to stop bleeding



**02**  
Apply tourniquet and tighten to stop bleeding



**03**  
Pull the hook loop tape tight and paste it back on



**04**  
Rotate the pressure fixed rod to tighten the strap



**05**  
Fasten enough to use the clasp to fix the rod



**06**  
Check for bleeding and distal pulse loss



**07**  
Attach the white hook loop tape



**08**  
Write down the time to prevent muscle necrosis caused by too much time



**We have 5000m<sup>2</sup> factory, our own factory**  
**The REAL STRENGTH is to show the true face to you**



# Strict certification testing provides quality guarantee and **WORRY-FREE EXPORT**



We have achieved some inspection certification, such as ISO9001:2015, BSCI, SGS, TUV, etc. and have won the title of “Guangdong Province Credible Enterprise” for three consecutive years. In addition, we have obtained over 20 product patents and trademark, as well as 5 patent evaluation reports. A series of quality inspection reports provide quality guarantee and worry-free export.

## CONTACT INFORMATION

TEL : +80 20 34992808  
+86 18026363923

Web : [www.homecommoditysupplier.com](http://www.homecommoditysupplier.com)

E-mail : [sales02@balance-china.com](mailto:sales02@balance-china.com)

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,  
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470