

Professionele groothandel

Tourniquet voor het aanbod van eerste hulp en noodhulpmiddelen

Productdetail

Productnaam	Tourniquet
Kleur	Zwart
Maat	3.8*65cm, 3.8*75cm, 3.8*95cm
Materiaal	Haaklusband & plastic/mentale staaf
Moq	Accepteer kleine bestelling
Pakket	1 pc/OPP -tas of aangepaste pakket
Aanpassing	Aangepast logo, verpakking en kleur kunnen worden gemaakt volgens gebruikersvereisten
Functie	Toegepast met één hand, snel en praktisch, toevallig schrappen, sneden en ander goed effect van hemostase.

Tourniquet is een gemeenschappelijke levensreddende apparatuur, die is gemaakt van hoogwaardige materialen en kan met één hand worden bediend, waardoor de bloedstroom volledig naar de ledematen wordt geblokkeerd door ernstige bloedingstrauma-wonden. Het is een geweldige EHBO -tool met een breed scala aan toepassingsscenario's, niet alleen in gevechten en medische redding, maar ook in buiten-, avontuur- en arteriële bloedingen. Een must-have voor iedereen!

CAT ROTARY TOURNIQUET



Easy to use



Breathable
&
Comfortable



Effective
Hemostasis



Compact
&
Convenient





FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response



RECORD USAGE TIME

Write down the time of use to easily determine the length of time



TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control



PRODUCT USAGE



**Wilderness
Combat**



**Outdoor
Adventure**

PREPAREDNESS AVERTS PERIL



**Bleeding
from injury**



**Hiking and
camping**

— HOW TO USE —



01
Tie the tourniquet three fingers' width above the wound to stop bleeding



02
Apply tourniquet and tighten to stop bleeding



03
Pull the hook loop tape tight and paste it back on



04
Rotate the pressure fixed rod to tighten the strap



05
Fasten enough to use the clasp to fix the rod



06
Check for bleeding and distal pulse loss



07
Attach the white hook loop tape



08
Write down the time to prevent muscle necrosis caused by too much time

Bedrijfsinformatie



CONTACT INFORMATION

TEL : +80 20 34992808
+86 18026363923

Web : www.homecommoditysupplier.com

E-mail : sales02@balance-china.com

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470