

# Professional Wholesale Tourniquet For First Aid And Emergency Medical Supplies Supply

## Product Detail

<b>Product name</b>	<b>Tourniquet</b>
<b>Color</b>	Black
<b>Size</b>	3.8*65cm, 3.8*75cm, 3.8*95cm
<b>Material</b>	Hook loop tape & plastic/mental rod
<b>MOQ</b>	Accept small order
<b>Package</b>	1 pc/opp bag,or customized package
<b>Customization</b>	Customized logo, packaging and color can be made according to user requirements
<b>Function</b>	Applied with one hand, fast and practical, accidental scrapes, cuts and other good effect of hemostasis.

**Tourniquet is a common life-saving equipment, which is made of high-quality materials and can be operated with one hand, completely blocking the blood flow to the limbs from severe bleeding trauma wounds. It is a great first aid tool with a wide range of application scenarios, not only in combat and medical rescue, but also in outdoor, adventure and arterial bleeding. A must-have for everyone!**

---

# CAT ROTARY TOURNIQUET



Easy to use



Breathable  
&  
Comfortable



Effective  
Hemostasis



Compact  
&  
Convenient







# FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



# HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response





# RECORD USAGE TIME

Write down the time of use to easily determine the length of time



# TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control





## PRODUCT USAGE



**Wilderness  
Combat**



**Outdoor  
Adventure**

## PREPAREDNESS AVERTS PERIL



**Bleeding  
from injury**



**Hiking and  
camping**



# — HOW TO USE —



**01**  
Tie the tourniquet three fingers' width above the wound to stop bleeding



**02**  
Apply tourniquet and tighten to stop bleeding



**03**  
Pull the hook loop tape tight and paste it back on



**04**  
Rotate the pressure fixed rod to tighten the strap



**05**  
Fasten enough to use the clasp to fix the rod



**06**  
Check for bleeding and distal pulse loss



**07**  
Attach the white hook loop tape



**08**  
Write down the time to prevent muscle necrosis caused by too much time

---

## Company Information



# CONTACT INFORMATION

TEL : +80 20 34992808  
+86 18026363923

Web : [www.homecommoditysupplier.com](http://www.homecommoditysupplier.com)

E-mail : [sales02@balance-china.com](mailto:sales02@balance-china.com)

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,  
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470