# Torniquete Médico

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**Single Route Buckle**

# RECORD USAGE TIME

Write down the time of use to easily determine the length of time

# TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control

# PRODUCT USAGE

Wilderness  
Combat

Outdoor  
Adventure

## PREPAREDNESS AVERTS PERIL

Bleeding  
from injury

Hiking and  
camping

# FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size

# HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response

# — HOW TO USE —

**01**  
Tie the tourniquet three fingers' width above the wound to stop bleeding

**02**  
Apply tourniquet and tighten to stop bleeding

**03**  
Pull the hook loop tape tight and paste it back on

**04**  
Rotate the pressure fixed rod to tighten the strap

**05**  
Fasten enough to use the clasp to fix the rod

**06**  
Check for bleeding and distal pulse loss

**07**  
Attach the white hook loop tape

**08**  
Write down the time to prevent muscle necrosis caused by too much time

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**We have 5000m<sup>2</sup> factory, our own factory**  
**The REAL STRENGTH is to show the true face to you**

# Strict certification testing provides quality guarantee and **WORRY-FREE EXPORT**

We have achieved some inspection certification, such as ISO9001:2015, BSCI, SGS, TUV, etc. and have won the title of “Guangdong Province Credible Enterprise” for three consecutive years. In addition, we have obtained over 20 product patents and trademark, as well as 5 patent evaluation reports. A series of quality inspection reports provide quality guarantee and worry-free export.

## CONTACT INFORMATION

TEL : +80 20 34992808  
+86 18026363923

Web : [www.homecommoditysupplier.com](http://www.homecommoditysupplier.com)

E-mail : [sales02@balance-china.com](mailto:sales02@balance-china.com)

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,  
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470