

Профессиональные оптовые турнике для первой помощи и поставки скорой медицинской помощи

Деталь продукта

Название продукта	Жгут
Цвет	Черный
Размер	3,8*65 см, 3,8*75 см, 3,8*95 см
Материал	Лента петли и пластик/ментальный стержень
Могил	Принять небольшой заказ
Упаковка	1 ПК/сумка ОРР или индивидуальный пакет
Настройка	Индивидуальный логотип, упаковка и цвет могут быть сделаны в соответствии с требованиями пользователя
Функция	Применяется одной рукой, быстрыми и практичными, случайными царапинами, разрезами и другим хорошим эффектом гемостаза.

Tourniquet-это обычное жизненно важное оборудование, которое изготовлено из высококачественных материалов и может работать одной рукой, полностью блокируя кровотоки до конечностей от тяжелых травм. Это отличный инструмент первой помощи с широким спектром сценариев применения, не только в бою и медицинском спасении, но и на открытом воздухе, приключениях и артериальном кровотечении. Обязательный для всех!



CAT ROTARY TOURNIQUET



Easy to use



Breathable
&
Comfortable



Effective
Hemostasis



Compact
&
Convenient





Hook loop tape

TIME:

Time recording tape

Pressure fixed rod

Single Route Buckle

FREE ADJUSTING BUCKLE

Adjustable to fit arm and leg size



HIGH-VISIBILITY RED TIP

In case of emergency, the red tip can immediately located and identified to make quick first aid response



RECORD USAGE TIME

Write down the time of use to easily determine the length of time



TIGHTEN THE PRESSURE FIXED ROD

Rotatable internal band for quick bleeding control



PRODUCT USAGE



**Wilderness
Combat**



**Outdoor
Adventure**

PREPAREDNESS AVERTS PERIL



**Bleeding
from injury**



**Hiking and
camping**

— HOW TO USE —



01
Tie the tourniquet three fingers' width above the wound to stop bleeding



02
Apply tourniquet and tighten to stop bleeding



03
Pull the hook loop tape tight and paste it back on



04
Rotate the pressure fixed rod to tighten the strap



05
Fasten enough to use the clasp to fix the rod



06
Check for bleeding and distal pulse loss



07
Attach the white hook loop tape



08
Write down the time to prevent muscle necrosis caused by too much time

Информация о компании



CONTACT INFORMATION

TEL : +80 20 34992808
+86 18026363923

Web : www.homecommoditysupplier.com

E-mail : sales02@balance-china.com

Address : No.2 Shengyuan Street, Beiliu Ind Park, Dagang Town,
Nansha District, Guangzhou, Guangdong, China

Zip code: 511470